



PROTOCOL FOR USE OF REPOSE FOOTCARE SOLUTIONS

Repose Foot Protector, Foot Protector Plus, Wedge and Sole Protector.

TREATMENT OBJECTIVES

- ✓ Prevention of initial ulcer/wound
- ✓ Maintain skin integrity
- ✓ Prevention of deterioration of existing ulcer wound
- ✓ Prevention of developing additional ulcer/wound

LIMITED MOBILITY

Patient mostly in bed, ability to sit out or go to the bathroom, shares a bed with partner

Assess for risk of falls
Needs of partner interaction
Seating arrangements/sitting out options

OPTION 1

Fit device worn by patient for in bed protection. Removable if patient becomes mobile.

OPTION 2

Fit static option(s) not worn by patient for in bed use.

OPTION 3

Consider sitting out protection

IMMOBILITY

Patient in bed

Assess movement/agitation in bed

OPTION 1

Fit device worn by patient, dependent on assessment of movement

OPTION 2

Fit static option(s) not worn by patient for in bed use.

REPOSE FOOTCARE SOLUTIONS

Repose Foot Protector /
Repose Foot Protector Plus

Repose Wedge and Repose Sole Protector in conjunction

Repose Wedge



OR



REPOSE FOOTCARE SOLUTIONS

Repose Foot Protector/
Repose Foot Protector Plus

Repose Wedge and
Repose Sole Protector
in conjunction



OR



ORDER CODES

Product	Dimensions	NHS Supply Chain Code	Repose Order Code
Repose® Sole Protector	L860mm X W 370mm	FER13070	6930001
Repose® Wedge	L 690mm X W 450mm	FER8136	6711100
Repose® Foot Protector (Pair)	L 380mm x W 200mm x H 180mm	FER731	6501100
Repose® Foot Protector Plus (Pair)	L 380mm x W 200mm x H 180mm	ELY591	6511100

FOR MORE INFORMATION

For more information please call **01495 235800**, email info@reposedirect.com or visit the Repose website.

REPOSEDIRECT.COM

**REPOSE CAN BE
CLEANED & REUSED**

