

## Repose<sup>®</sup> Trolley Mattress

Order Code: 6601100




### An effective aid in the prevention and treatment of pressure ulcers

Repose is an effective aid in the prevention of pressure ulceration for all patients including those assessed as very high risk, and for the treatment of existing category 1 and 2 pressure ulcers. Clinical supervision is advised if damage is severe.

#### To Inflate

- 1 Remove Repose from the tube, refit the two tubes together to form a pump
- 2 Connect the valve on the pump to the valve on the device - Push to fit
- 3 Inflate Repose until you hear air exhausting from the pump valve. Discontinue pumping and wait until you no longer hear air escaping
- 4 Cover the valve on the device using the solid end of the valve cap. Turn over with valve underside and away from patient. Secure with straps

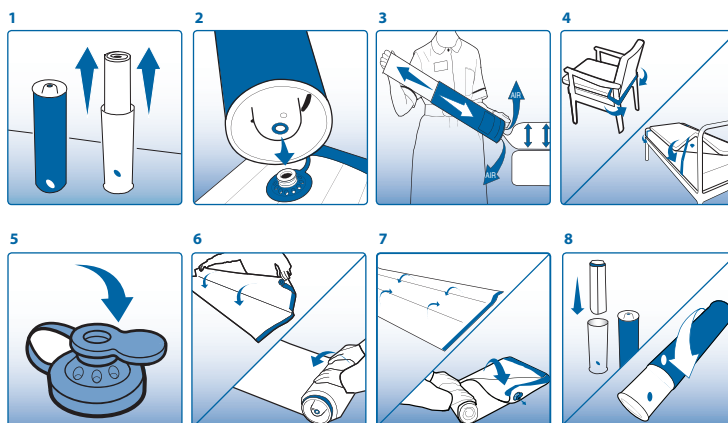

#### To Deflate

- 5 Insert the open end of the valve cap into the device valve. Air can now escape
- 6 Fold and roll up the device starting at the point furthest away from the valve until all air is expelled. Clean and allow to dry
- 7 Re-fold, and re-roll as tightly as possible
- 8 Replace device into the pump. Refit the two tubes together and rotate outer tube until it clicks closed

#### Please note, Repose is not suitable for the following:

- Persons weighing in excess of 139kg
- Persons with unstable spinal fractures
- Where body shape is such that the person cannot be fully supported by the Mattress or Cushion

The Repose Mattress Overlay is suitable for use on top of a bed or trolley mattress. Use side rails in accordance with manufacturers instructions and appropriate risk assessment.

Check device daily to ensure proper inflation level. In case of doubt, repeat inflation procedure with the user off the product.

Keep away from babies, small children and pets. Avoid contact with heat and sharp objects

**If deterioration in skin condition is noted, seek clinical advice**

#### REPOSE PRODUCTS CAN BE CLEANED AND REUSED



WIPE CLEAN



DO NOT DRY CLEAN



DO NOT TUMBLE DRY



DO NOT IRON

REPOSE PRODUCTS CAN BE CLEANED WITH SOAP OR DETERGENT AND WATER OR IN ACCORDANCE WITH LOCAL INFECTION CONTROL POLICY. RINSE WITH CLEAN WATER AFTER USING DISINFECTANT. ALLOW TO DRY BEFORE USE OR STORAGE. REPOSE IS SUPPLIED WITH A MANUFACTURERS WARRANTY OF 24 MONTHS FROM DATE OF DISPATCH.

#### IMPORTANT

CHECK PRODUCT DAILY TO ENSURE CORRECT INFLATION LEVEL. IN CASE OF DOUBT, REPEAT INFLATION PROCEDURES WITH THE PATIENT OFF THE PRODUCT. KEEP AWAY FROM SHARP OBJECTS AND DIRECT HEAT.